


Interpreting Results


Results will be displayed in the result window:

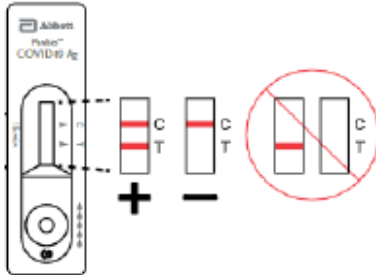


Legend:

C = Control line


T = Test line

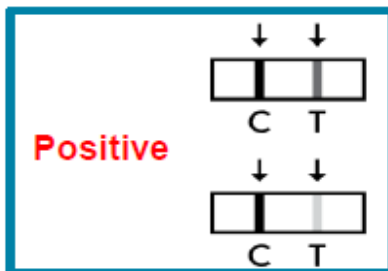

Positive = the presence of the **control line (C)** and the **test line (T)** in the result window. The presence of any test line (T), no matter how faint, indicates a positive result.

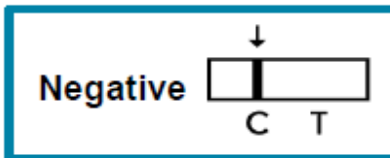

Negative = the presence of **only the control line (C)** and no test line (T) within the result window.

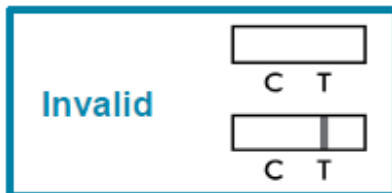

Invalid = if the control line (C) is not visible.

Note: Results are also invalid if more than 20 minutes has elapsed.

<p>"Invalid" result</p>	<p>If test results are invalid, the test must be repeated with a new swab collected. The same cartridge that received an invalid result should not be re-used. If a second invalid test result is obtained, stop staff testing until the cause of the failures is identified. Contact distributor if problems persist.</p>
-------------------------	--

Troubleshooting